ALHYDRAN

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EN

Name: ALHYDRAN, medical hydrating cream.

Product description: Medical hydrating cream. Ensures optimal, long-lasting hydration and prevents moisture loss. Ideal for treatment of disrupted skin barrier. Suitable for (after)care of burn wounds, treatment and prevention of scars, treatment of (extremely) dry skin and reduction of redness and itching.

ALHYDRAN is hypoallergenic and contains no parabens or fragrances. Suitable for all skin types.

Ingredients: Aloe Barbadensis Leaf Juice (Aloe Vera Gel from Aruba), Aqua (Water), Cetearyl Alcohol, Caprylic/ Capric Triglyceride, Decyl Oleate, Glyceryl Stearate, Cetearyl Isononanoate, Sorbitan Stearate, Ceteareth-20, Propylene Glycol, Phenoxxyethanol, Ceteareth-12, Cetyl Palmitate, Caprylyl Glycol, Chlorphenesin, Tocopheryl Acetate (Vitamin E Acetate), Ascorbic Acid (Vitamin C), Simmondsia Chinensis Seed Oil (Jojoba Oil), Disodium EDTA.

Properties: ALHYDRAN is a medical cream with a base of freshly manufactured Aloe Vera Gel. Aloe Vera gives the cream strong moisturising properties and a mildness particularly suitable for damaged and vulnerable skin.

The other ingredients of the ALHYDRAN product line contain a high level of high-end cosmetic oils and fats, which further contribute to the general moisturising characteristics of the product. ALHYDRAN makes the skin supple and elastic, and is particularly suitable for skin types with reduced elasticity. The product is extremely mild and tolerated by a wide range of skin types. ALHYDRAN does not spot, and is water-washable.

Indications/uses:

- Aftercare of recently healed skin following acute and chronic wounds.
- Aftercare of damaged skin, in order to prevent scarring.
- Improves the condition of existing scars resulting from burn wounds.
- Prevents and reduces itching and pain immediately after skin has healed closed.
- Reduces eczema symptoms.
- Aftercare of burns, in particular skin care, immediately before and after wearing pressure garments and/or silicone dressings.
- Aftercare of wounds caused by freezing.
- Preventive skin care during oncological and radiation therapy.
- Aftercare of skin grafts and skin transplantation donor sites.
- Preventive care of red/dry skin and stretch marks.
- Aftercare following chemical peeling and laser treatment.
- Aftercare following medical and cosmetic surgery.

How to use:

- Apply a thin, even layer of ALHYDRAN three times a day - in the morning, at midday and in the evening.
- ALHYDRAN may be used without limitation (for years, if required).

Do not use:

- If there is an allergic reaction to one of the ingredients.
- On open wounds, or skin perforated by stitches that has not yet healed.
- On infected skin.
- On chronic rash or other irritated skin.
- Do not use on children under the age of 3 months.

Special precautions:

- Keep out of reach of children.
- External use only.
- Use only on healed, closed skin and not on open wounds.
- Avoid direct contact with eyes and mucous membranes.
- Do not use if packaging has been damaged or opened in transit.
- When using on children under 6 months, please consult your physician first

Side-effects:

There are no known side effects to date. Any serious incidents that have occurred in relation to the device should be reported to the manufacturer and the competent authority (of the member state) in which the user and/or patient is established.

Storage:

- See pack for expiry date.
- Store at temperatures not exceeding 25°C.
- The product may slightly discolour or change consistency after opening due to the high concentration of natural ingredients - which include fresh, Aloe Vera gel. This does not affect the proper functioning of the cream.

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